

# BOLERO

**Medical Privacy  
Screen**

**Assembly Instructions**



**Model:  
Y188**

## Introduction

The Privacy Screen is quality checked and tested before leaving the factory. However it is important you check that all parts are included in the package, and that no damage has occurred to the Privacy Screen whilst in transit. Check that all components are correctly fastened before use.

The package contains:

1. Four detachable castor sets
2. Assembled four fold screen

Remove all packaging, and dispose of it in accordance with local waste regulations.

## Assembly

Attach the detachable castor set (2) onto the base of the screen. Ensure that the button on the frame bottom of the screen lines up with the hole on the castor base, and clearly comes through.



Repeat this for all four sections of the screen (note that the castor sets for screens 1 and 3 are at right angles to the castor sets for 2 & 4)

## Care of the Screen

To keep the screen clean, simply wipe with a damp cloth or anti-bacterial wipes.

Always ensure that the screen is left dry.



<b>EN</b>		Fourth Way, Avonmouth, Bristol, BS11 8TB United Kingdom
<b>Eire</b>	+44 (0)845 146 2887	Unit 9003, Blarney Business Park, Blarney Co. Cork, Ireland
<b>NL</b>	040 - 2628080	
<b>FR</b>	01 60 34 28 80	
<b>BE-NL</b>	0800-29129	

<b>BE-FR</b>	0800-29229	
<b>DE</b>	0800 - 1860806	
<b>ES</b>	901-100 133	
<b>AU</b>	1300 225 960	15 Badgally Road, Campbelltown, 2560, NSW, Australia

